1
2018

HYVÄT KÄDET

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Viitteet:

1. WHO Guideline Steering Group, Guidelines Development Group, Systematic Reviews Expert Group, External Review Group
2. Adriana Cristina de Oliveira, Camila Sarmento Gama. Evaluation of surgical glove integrity during surgery in a Brazilian teaching hospital. *American Journal of Infection Control*. Osa 42, nro 10, lokakuu 2014, 1093–1096.
3. Twomey C, Reiss L. Does Double Gloving Double the Protection? A Look at the Issues. *Infection Control Today*, 30. huhtikuuta, 2000. <https://www.infectioncontroltoday.com/hand-hygiene/infection-control-today-does-double-gloving-double-protection>
4. Spruce L, Back to Basics: Sterile Technique, *AORN Journal* 105 (toukokuu 2017), 478–487.
5. Personal communication from nursing and surgical staff
6. Personal protective equipment for use in a filovirus disease outbreak: Rapid advice guideline WHO 2016 <http://apps.who.int/iris/bitstream/handle/10665/251426/9789241549721-eng.pdf?sequence=1&ua=1>
7. Viite: WHO Emergency and Essential Surgical Care http://www.who.int/surgery/challenges/esc_infectious_diseases/en/
8. Surgical Care at the District Hospital WHO 2003 <http://www.who.int/surgery/publications/en/SCDH.pdf?ua=1>
9. WHO/World Alliance for Patient Safety, Information Sheet 6, Glove Use (Technical) <http://www.who.int/gpsc/tools/InfSheet6.pdf?ua=1>
10. Glove Use Information Leaflet: Outline of the evidence and considerations on medical glove use to prevent germ transmission. Tarkistettu elokuussa 2009 http://www.who.int/gpsc/5may/Glove_Use_Information_Leaflet.pdf
11. WHO Guidelines on Hand Hygiene in Health Care: First Global Patient Safety Challenge - Clean Care is Safer Care. WHO Patient Safety 2009 ISBN 978 92 4 159790 6. http://apps.who.int/iris/bitstream/handle/10665/44102/9789241597906_eng.pdf;sequence=1
12. Kourtis A P et al. HIV–HBV Coinfection — A Global Challenge. *N Engl J Med*. 10. toukokuuta 2012; 366(19): 1749–1752. <https://www.ncbi.nlm.nih.gov/pmc/articles/PMC4453872/>
13. <http://www.pewglobal.org/2018/03/22/at-least-a-million-sub-saharan-africans-moved-to-europe-since-2010/>
14. WHO Regional Office for Europe: Migration and health: key issues. <http://www.euro.who.int/en/health-topics/health-determinants/migration-and-health/migrant-health-in-the-european-region/migration-and-health-key-issues>
15. HSE: <http://www.hse.gov.uk/biosafety/blood-borne-viruses/use-of-gloves.htm>
16. Standard Infection Control Precautions Literature Review: Personal Protective Equipment (PPE)
17. <http://www.nipcm.scot.nhs.uk/documents/sicp-ppe-gloves/>
18. Tanner J, Parkinson H, Tanner J, Parkinson H. Double gloving to reduce surgical cross-infection. [Update of Cochrane Database Syst Rev. 2002;(3):CD003087; PMID: 12137673]. *Cochrane Database Syst Rev* 2006;3:CD003087. <http://cochranelibrary-wiley.com/doi/10.1002/14651858.CD003087.pub2/full>
19. Esimerkiksi: Laine T, Aarnio P. How often does glove perforation occur in surgery? Comparison between single gloves and a double-gloving system. *Am J Surg*, kesäkuu 2001, 181(6): 564–6.
20. Naver LPS. Incidence of glove perforations in gastrointestinal surgery and the protective effect of double gloves: A prospective, randomised controlled study. *Eur J Surg* 2000;166(4):2000.
21. Thomas S, Agarwal M, Mehta G, Thomas S, Agarwal M, Mehta G. Intraoperative glove perforation—single versus double gloving in protection against skin contamination. *Postgrad Med J*, heinäkuu 2001, 77(909): 458–60.
22. Tanner J, Parkinson H. Double gloving to reduce surgical cross-infection. *Cochrane Database of Systematic Reviews* 2006, Issue 3. Art.no: CD003087. DOI: 10.1002/14651858.CD003087. Pub 2. Päivitetty 28.7.2009.
23. Childs T. Use of Double Gloving to Reduce Surgical Personnel's Risk of Exposure to Bloodborne Pathogens: An Integrative Review *AORN J* 98 (December 2013) 586–593. https://www.aorn.org/websitedata/cearticle/pdf_file/CEA13534-0001.pdf
24. Bulletin of The American College of Surgeons. Revised statement on sharps safety October 1, 2016 <http://bulletin.facs.org/2016/10/revised-statement-on-sharps-safety/#.WxAMH2CWzcs>
25. Surgical site infections: prevention and treatment Clinical guideline [CG74] Julkaistu: Lokakuu 2008. Päivitetty: Helmikuu 2017. <https://www.nice.org.uk/guidance/CG74/chapter/1-Guidance#intraoperative-phase>
26. Moucha CS. Safety First: Infection prevention tips *AAOS Now*, lokakuu 2013 <https://www.aaos.org/AAOSNow/2013/Oct/clinical/clinical5/?ssopc=1>
27. Extra gloves or special types of gloves for preventing sharps injuries in healthcare workers http://www.cochrane.org/CD009573/OCCHEALTH_extra-gloves-or-special-types-of-gloves-for-preventing-sharps-injuries-in-healthcare-workers

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©2018. Mölnlycke Health Care Oy. Kaikki oikeudet pidätetään.